


HUUMAN

Understanding Face Masks

ANATOMY OF FACE MASK


The Huuman disposable face mask is a light and breathable 3-layer mask ideal for preventing the spread of disease. Masks are fluid resistant and intended to provide the wearer protection against larger droplets, splashes or sprays of bodily or other hazardous fluids. The face mask is also effective in reducing exposure to smaller particles such as dust, pollution, smoke and other micro containments.

Not aseptic. Nonsterile. Not made with natural rubber latex.

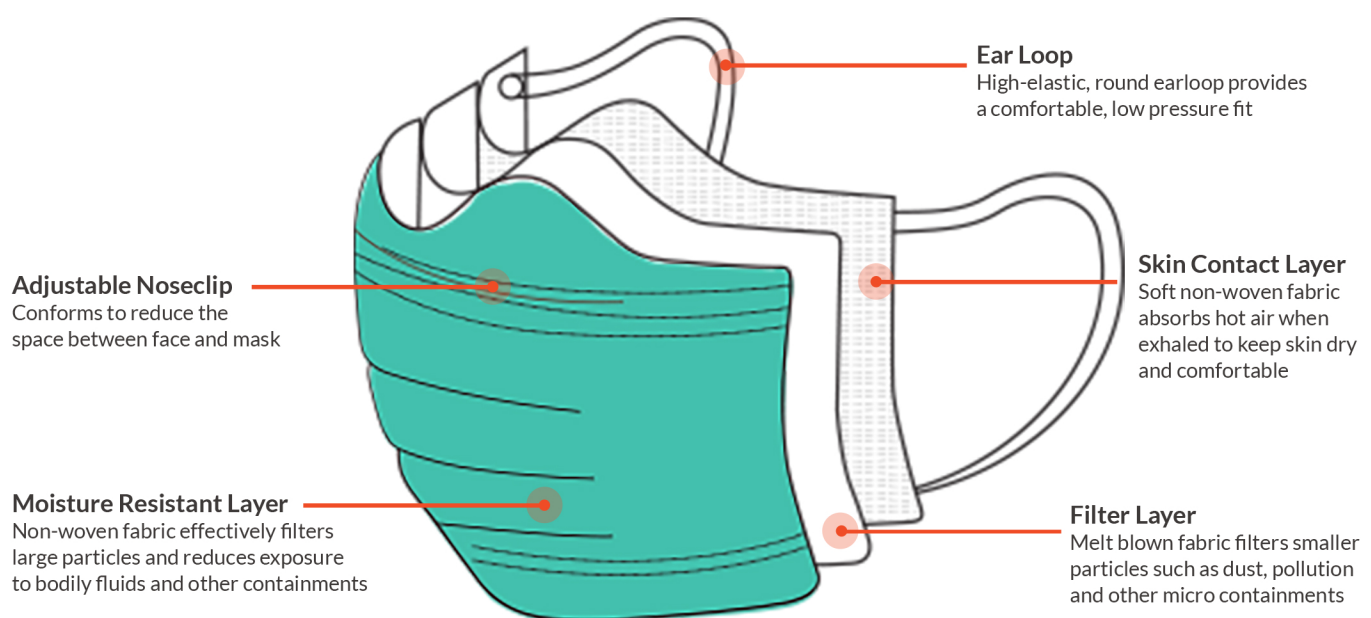

HOW TO PUT ON

1. Clean hands with soap and water or an alcohol-based hand sanitizer and inspect the mask for any obvious tears or holes before applying the face mask.
2. Determine the correct orientation of the mask by determining which side is the front (this is the colored side) and which side is the top (this is the side with the adjustable nose clip) of the mask.
3. Hold the mask by the ear loops and place over ears.
4. Adjust nose piece to the shape of your nose for a comfortable fit.
5. Pull the bottom of the mask over face and chin.

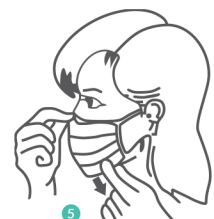

HOW TO REMOVE

Masks should be replaced whenever they become moist or visibly contaminated

1. Clean hands with soap and water or hand sanitizer before removing the mask.
2. Hold both ear loops and gently lift and remove the mask.
Do not touch the front of the mask - it is contaminated.
7. Dispose of the mask in the trash. Do not reuse.
8. Clean your hands with soap and water or hand sanitizer.

WHEN A FACE MASK SHOULD BE WORN

A face mask should be worn under the following conditions to reduce the spread of illness:

- If you are sick with a cough or sneezing illness (with or without a fever).
- When caring for or coming in contact with a person with a known or suspected illness such as the 2019-nCoV infection.
- Before entering a public place or anywhere people congregate, such as a grocery store.